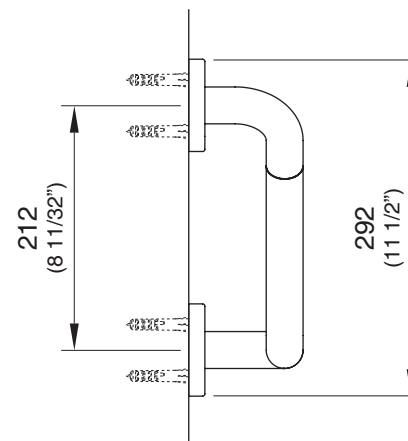
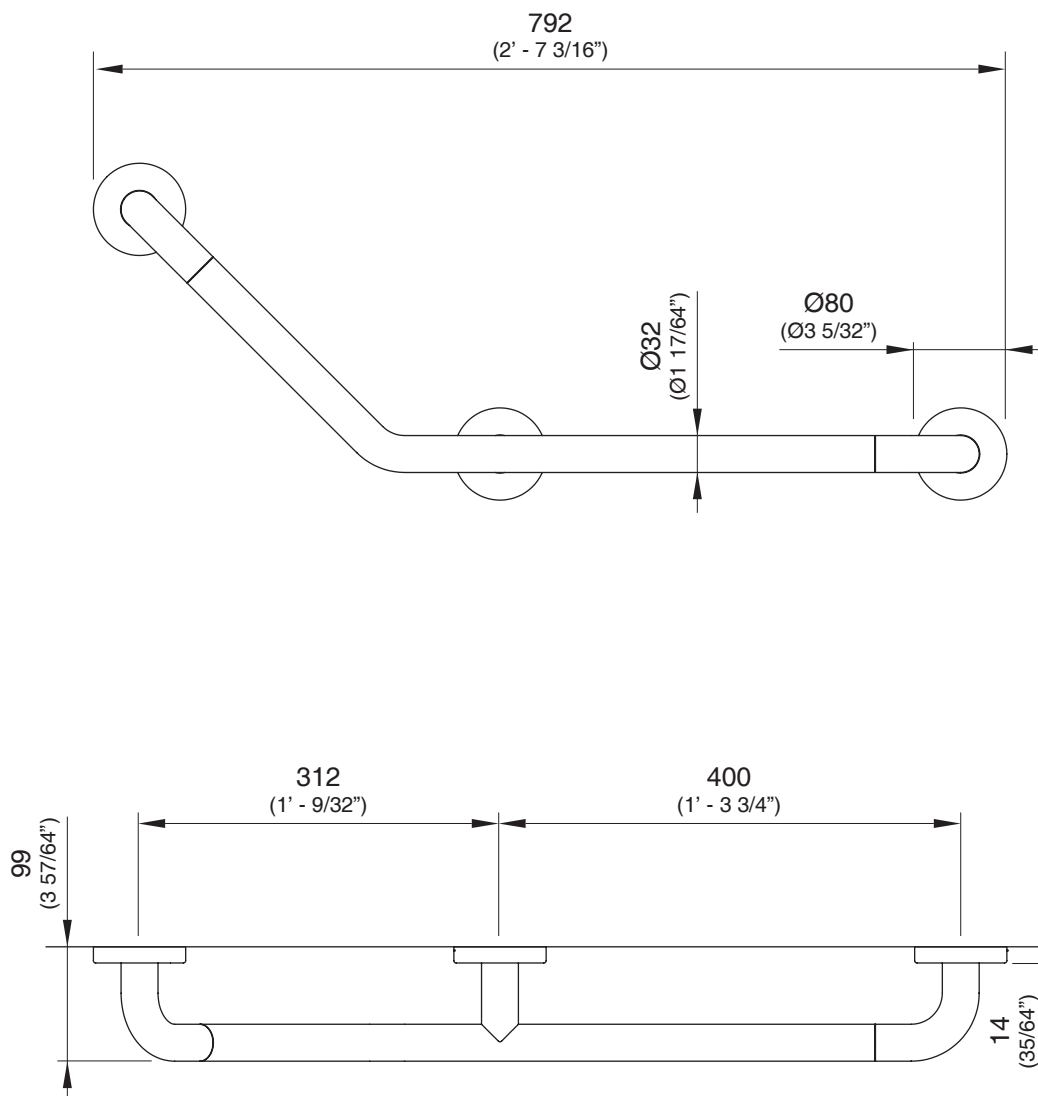
HDL03 _REV10

HDL03 / SAFETY GRAB BARS

AISI 316L STAINLESS STEEL

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Angular safety grab bar, installation on the left

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CEADDESIGN SRL s.u. has the property rights on these drawings, therefore they cannot be used and/or transferred to third party, even in partial form, without any prior authorization. Furthermore CEADDESIGN SRL s.u. reserves the right to apply any change to these drawings in order to upgrade the product.

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TECHNICAL WARNINGS

This product is comply with the directive 93/42/EEC for medical device.

The safety grab bar offer support to elderly or disabled people, with or without the help of an assistant, and must not be fitted inside saunas or extremely hot areas. Do not use the product for any use other than those shown in this instructions.

The application of a load greater than 150 kg can cause permanent damage. In case of a visible permanent deformation please contact CEADESIGN SRL.

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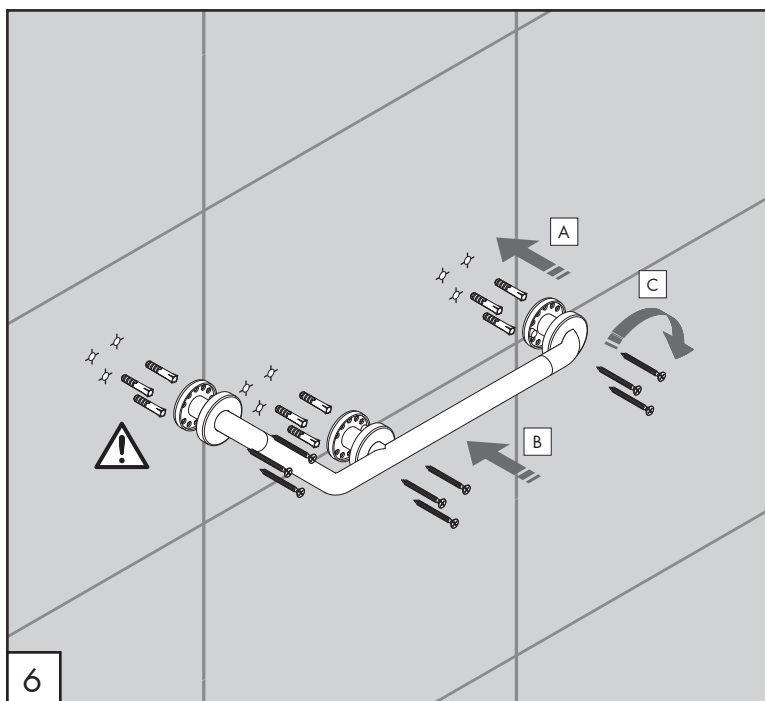
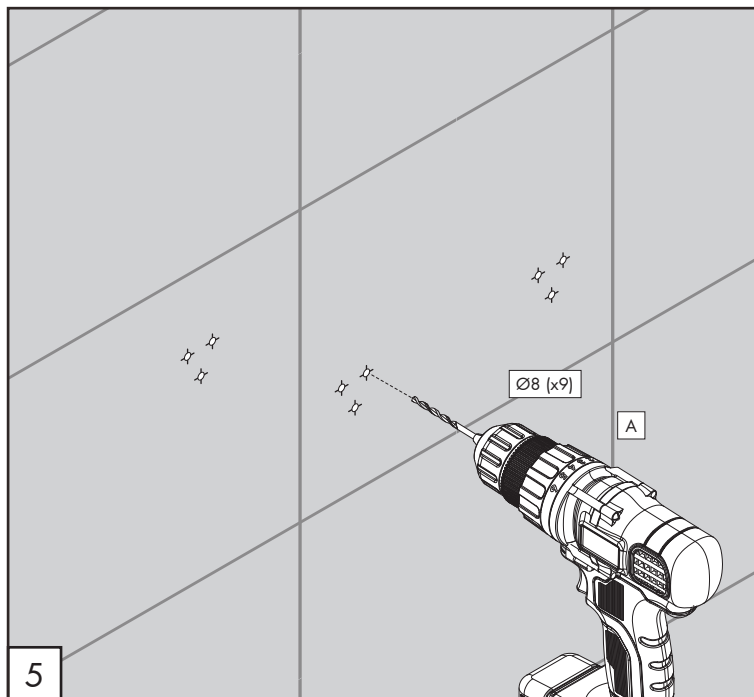
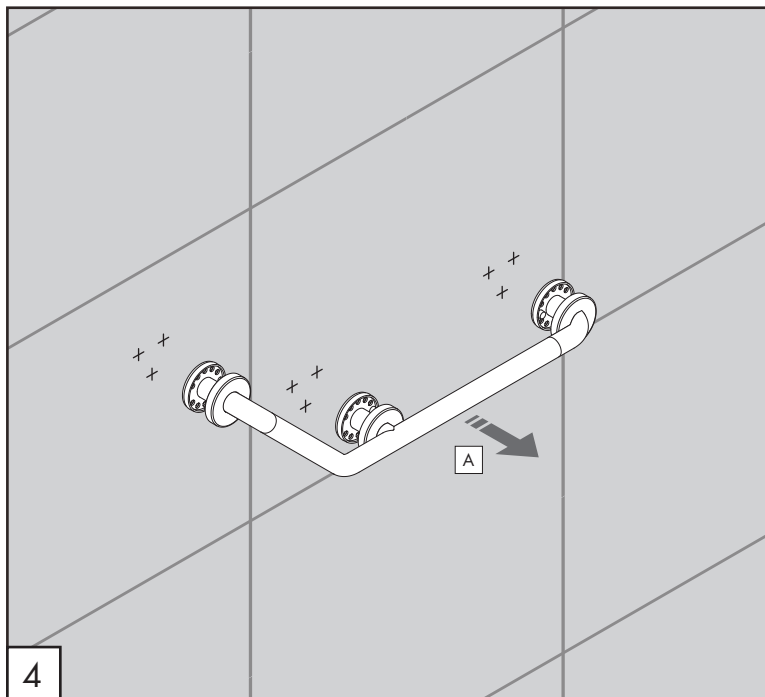
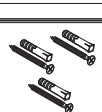
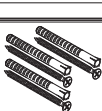
TECHNICAL WARNINGS

The safety grab bar can be fixed on concrete, natural stone, solid or sand-lime brick, hollow brick masonry, plasterboard and wooden plate. **For fixing on plasterboard and wooden plate, provide a reinforcement panel in plywood or solid fir with a minimum thickness of 25 mm.**

The safety grab bar must never be tampered with or modified. Check periodically (once a month) that the safety support has not been damaged or worked loose. Failure to regard these precautions may cause serious damage or accidents.

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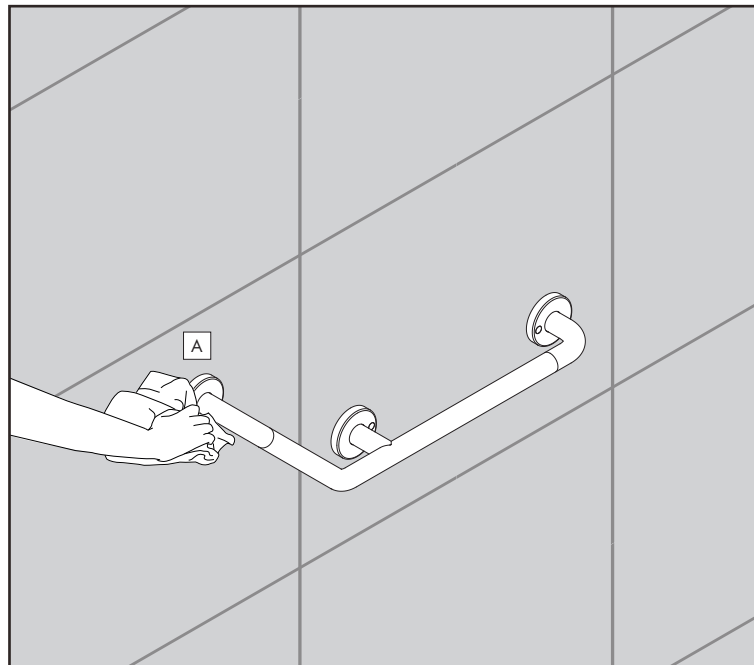
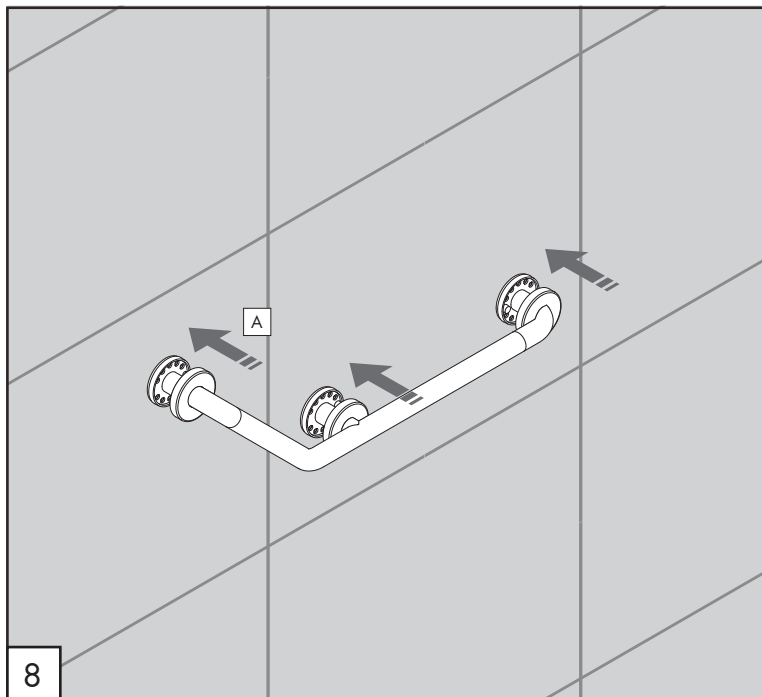
TECHNICAL WARNINGS

Clean the outside surface with a soft damp cloth. Do not use scouring pads or brushes of any kind or abrasive detergents containing microscopic particles that can scratch the surfaces and ruin the satin and polished surface finishing. Do not use sanitizers such as bleach, chlorides, ethylene oxides, nonionic surfactants that could damage the surface of the products. Do not use detergents based on chloric, hydrofluoric, phosphoric and muriatic acid.

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